

WOW<sup>®</sup>  
*fusion*

HYALURONIC ACID SERUM  
HAFACITOR<sup>1</sup>

Base serum for  
hydration and  
skin restoration

## HAFACITOR<sup>1</sup>

HYALURONIC ACID SERUM

### INSTRUCTIONS AND INFORMATION

#### PRESENTATION

5 x 5 ml Vials

#### INGREDIENTS – INCI

Sodium Hyaluronate 1%

Sodium Chloride 0.9%

Sterile Water for Injection

Added up to 5ml

#### INDICATIONS

High quality high molecular weight Hyaluronic Acid serum. For hydration, collagen and elastin stimulation and the overall improvement of the skin texture and tone. To be used as a base carrier for other WOW fusion<sup>®</sup> solutions. Intended for topical use with the WOW fusion<sup>®</sup> device.

#### CONTRAINDICATIONS

It must not be used on:

- Patients who have a skin disease or condition
- Patients having immunotherapy
- Patients who have hypersensitivity or a known allergy to one of the components
- High risk of keloid scarring
- High risk of post inflammatory hyperpigmentation
- Pregnant or breastfeeding
- Under 18s

#### CAUTIONS

Caution must be taken:

- Blood thinning medication
- Multiple Sclerosis
- Lupus
- Uncontrolled diabetes
- Herpes Simplex

#### PRECAUTIONS FOR USE

- No clinical data available with respect to the tolerability of HA FACTOR 1 in an area that has already been treated with another cosmetic product for deep topical use
- It is recommended that the patient should not apply make-up for 4 hours after treatment
- Avoid prolonged exposure to the sun and the use of sunbeds for 4 weeks
- No use of saunas or steam rooms 2 weeks post treatment

#### POSSIBLE SIDE EFFECTS

- Mild local redness that disappears after a few hours
- Itching
- Bruising
- Skin dryness
- Flakiness

If any inflammatory reactions or side effects have not disappeared after 3 days, the patient should inform the professional as soon as possible and the professional must report to Circa Skin Ltd. or their representatives in their country.

#### WARNING

- Always check the expiry date
- Keep out of the reach of children
- Do not expose to direct light
- Refrigerate after opening
- Use within 1 month

Manufactured by  
RSI 6728-PS  
for WOW facial<sup>®</sup>  
by Circa Skin Ltd.

207 Regents Street,  
London, W1B 3HH.

